



Pain, lameness and neuromuscular difficulties are no longer an issue: BT Hock is used in conditions that cause lesions in the region of the hock joint. The brace has three pockets in order to insert two rigid lateral splints and an adjustable fore splint.

It is used for symptoms such as pain, lameness or difficulty in moving.

The brace can also be used without splints for conditions such as arthritis and arthrosis. This is because the compression it provides increases the blood pressure in the area of the hock, thus raising the temperature. This reduces the pain caused by the conditions.

The fore splint is recommended for those cases in which it is necessary to completely block the hock joint (for example after surgery or because of neurological problems).

Used also for protection purposes, it can contain plasters or bandages, with or without medication, in the case of surgical wounds or pressure sores. Also excellent after surgery.

FEATURES

- ▶ Breathable fabric.
- ▶ Excellent wearability.
- ▶ The brace can be used in pre- and in post-surgery.
- ▶ Two rigid lateral splints and the adjustable fore splint are included with the brace.
- ▶ The splints can be removed by extracting them from their pockets.
- ▶ The fore splint is adjustable.

RECOMMENDATIONS FOR USE

- ▶ During the period of restraint, it is advisable not to leave the dog alone for too long, because it might try to remove the brace by biting or scratching it.
- ▶ **Caution:** direct contact between the splints and the skin means that the brace may cause reddening if the dog is taken out to play or run while wearing it. We recommend that this be prevented by placing some gauze or cotton wool between the splints and the skin.
- ▶ Consult a veterinarian when using rigid splints.
- ▶ Always remove the rigid splints before washing.

SIZES

Available in five adjustable sizes:

