

BT JOINT CARPAL BRACE



The carpal brace is used for conditions involving lesions to the carpus, leading to pain, lameness, or difficulty in moving. It provides great protection for bandages and plasters when treating surgical wounds (arthrodesis) or licking sores. It also gives great results in pre-operative care: acute or chronic arthritis (osteoarthritis, osteochondrosis and similar conditions) and injuries to the ligaments are no longer a problem.

The brace can also be used without splints for conditions such as arthritis and arthrosis. This is because the compression it provides increases the blood pressure in the area, thus raising the temperature and reducing the pain caused by the conditions.

Once the brace has been placed around the dog's leg, it is not necessary that the two white-trimmed edges meet, since the closure of the brace is by means of the blue straps and stabilization is provided by the splints on the back of the brace.

ADVANTAGES

Choosing the excellence of the BT Joint carpal brace means ensuring maximum stability of the injured area, with a consequent reduction in swelling. BT Joint promotes improved balance between the muscle groups in the carpal area, thus facilitating motor functions and also promoting healing.

FEATURES

- ▶ The brace can be used in pre- and in post-surgery.
- ▶ Breathable fabric.
- ▶ Excellent wearability.
- ▶ Hand washable.

RECOMMENDATIONS FOR USE

- ▶ During the period of restraint, it is advisable not to leave the dog alone for too long, because it might try to remove the brace, by biting or scratching it.
- ▶ Always remove the rigid splints before washing.

SIZES

Available in five adjustable sizes.

